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THE EXTRACTOR

Remove more venom.
Provides more suction-750 millibars.
No cutting, no scalpel necessary.
Safe for children.
Also helpful in treating the bites and
stings, hornets, mosquitoes.
Recommended medically as the only
acceptable first aid device for snakebites.

HOW DOES IT WORK?

The Extractor was designed specifically to
provide the most powerful suction available
to safely remove venoms and poisons.
It's suction of 750 millibars is so powerful
there is no cutting necessary.
This eliminates using and carrying the
dangerous scalpel blades or knives
recommended by less effective kits.
The Extractor removes the poisons which
cause the pain. The suction is created by
simply thrusting the plunger downward
using the thumb and two fingers.

CARE & CLEANING OF THE EXTRACTOR

Do not wash the extractor or get it wet!
This may destroy the vacuum-creating
mechanism in the extractor.
You should wash and sterilize the cups
after each use but do not wash the extractor.
Thoroughly cleaning the cups with soap and
water is usually adequate.
Treating with alcohol is best.
Do not boil the cups in water.

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HOW TO USE THE EXTRACTOR PUMP

The Extractor pump creates powerful suction designed to remove venom from the body by sucking it out the same cavity through which it was injected.

For snake bites DO NOT cut an "X" or suck out venom with the mouth. This is ineffective and increases trauma in the area of the wound. The venom could get into the blood stream of a rescuer who has even a small cut in the mouth. For bees, DO NOT pull a stinger with fingers or tweezers as this will squeeze more venom into the victim. Instead, carefully flick it out with the blunt side of a knife, a credit card or other non-sharp object.

1. Don't panic. Begin emergency treatment only after the victim is quiet. This will slow the body's absorption rate of the poison. The Extractors pump is most effective when used quickly after the bite.

2. If necessary to improve suction, use the safety razor provided to remove body hair from the bitten area. This should only be necessary for treating bites from snakes, spiders, or sting rays. Other techniques to improve suction include wetting or covering the surface under the suction cup with petroleum jelly. (Note: applies only to areas of heavy body hair.)

3. Fit your Extractor pump with the suitable size suction cup. The smallest cup may be fitted in either direction.

4. Pull the plunger out to its fullest extent and place the cup over the bite or sting.

5. Using your thumb, push the plunger all the way in until you feel the suction, and let the pump remove poison. This should take from 60-90 seconds for insect bites. The first five minutes are most beneficial for applying the pump to a snake bite. Center a suction cup over one fang hole at a time. Alternate between fang Notes even two-three minutes for 15 minutes. Discontinue use after 15 minutes.

6. Pull the plunger up gently to release the vacuum. If present, poison mixed with body fluids will be on the surface of your skin. Be careful. Avoid poison splatter. For snakebite, IF no venom appears on the skin's surface, and IF there is no swelling, and IF you were not bit by a coral snake, and IF you have performed steps 1-6 correctly:

- It may have been a non-venomous snake

- It may have been a dry bite (no venom injected)

However, if symptom appear within three to four hours seek medical attention immediately.

7. Wash the fluids away carefully from the wound with soap and water. Apply alcohol and cover wound with the bandage provided in your kit.

8. Treat for shock and give artificial respiration, if necessary.

9. Seek medical help immediately. Children and adults under 120 lbs. (55kg.) are particularly at risk, as is anyone who has received multiple bites, stings, or a full load of venom.

10. After the person bitten is properly cared for, and the danger has passed, wash the suction cup so as to remove all the fluid residue. When possible, sterilize the cups between uses, especially after exposure to snake venom. DO NOT wash the pump! This may destroy the vacuum-creating mechanism and the pump.

DO NOT use the pump in areas of sensitive skin such as eyelids, genitals, or ear cavities.